

# Art Paper 80 GSM

Front 155.00 mm

**Mankind**

# Rapid News™

FOR HOME DETECTION

रैपिड न्यूज़


## Male Fertility Test


INSTANT SPERM TEST KIT


**MEN HEALTH**


- + Easy to use
- + Instant results
- + Test from the convenience of your home

### PROCEDURE

 Step 1- Collect semen sample in a clean cup

 Step 2- Mix semen sample to the assay buffer vial

 Step 3- Put 2 drops of the diluted sample to the sample slot (S) on test card

 Step 4- Read the results within 5-10 minutes

- Step 1) Collect the semen sample in a clean and dry specimen collection cup.
- Step 2) Allow the sample to stand for 60 minutes, till the semen is fully liquified.
- Step 3) Open the pouch just before use.
- Step 4) Using the specimen applicator provided, transfer 0.1 ml of semen sample to assay buffer vial.
- Step 5) Mix it well. Open the cap of vial & transfer 2 drops of diluted sample to sample slot (S) marked on the test card.
- Step 6) Read the results within 5-10 minutes.
- Step 7) Discard the test card after use.

**Storage: Store at 2°C to 40°C. Protect from light & moisture. Do not freeze.**  
Keep out of reach of children.  
This product is not intended to cure or prevent any disease.  
For single use only.  
For in vitro diagnostic use only.  
Not for internal or medicinal use.  
**FOR EXTERNAL USE ONLY**

180.00 mm

Back

### Interpretation of Result


**Positive:** Two pink/purple coloured lines at C & T. It means that specimen has normal sperm count (greater than 15 million/ml).


**Note:** The intensity of colour at T will vary depending on the concentration of SP-10 protein present in the specimen. Therefore, any shade of colour in test line (T) should be considered normal.


**Negative:** One pink/purple coloured line at C. It means that specimen has abnormal sperm count (less than 15 million/ml).


**Invalid:** One pink/purple coloured line at T or No pink/purple coloured line at C & T.

The test should be repeated with a new test kit. This might occur when insufficient specimen (too much or too less) volume is used. Follow the instructions carefully.

 Positive

 Negative

 Invalid

 Invalid

**Presentation:** Rapid News Male Fertility Test Kit contains 1U Test Pouch, 1U Collection Cup, 1U Applicator, 1U Assay buffer vial are packed in a carton along with a package insert.

**Manufactured in India by:**  
Koshbio Private Limited  
Plot No. 39, Block-I, DLF Industrial Estate,  
Phase-1, Faridabad, Haryana-121003

**Marketed by:**  
**MANKIND PHARMA LTD.**  
208, Okhla Industrial Estate-3,  
New Delhi-110020  
Contact Manager, Customer Care Cell:  
Toll Free No.: 18001034400  
Email: contact@mankindpharma.com  
TM-Trade Mark Under Registration

MKA0TPOIOI1488000

OEM19-MFT01-IFU-01-00