


Art Paper 80 GSM

180.00 mm

Front — 155.00 mm —



Rapid News™

FOR HOME DETECTION


Menopause Test

INSTANT URINE TEST KIT


WOMEN
HEALTH

- + Easy to use
- + Instant results
- + Test from the convenience of your home


COMMON SIGNS & SYMPTOMS



Hot flashes



Mood changes



Unusual weight gain

Rapid News Menopause test is a rapid lateral flow chromatographic immunoassay for the qualitative detection of Follicle Stimulating Hormone (FSH) level in urine to evaluate the onset of menopause in women.

Menopause is normal condition experienced by all women around their 40's. It marks the end of occurrence of menstrual cycle.

FSH (Follicle stimulating hormone) is responsible for stimulating the ovaries to produce eggs and also for maintaining fertility. During menopause, FSH levels are higher than normal, which causes various symptoms and changes in women's body.

PROCEDURE

- Step 1) Open the pack just before use.
- Step 2) Collect the urine sample in a clean & dry container.
- Step 3) Add 2-3 drops of urine in the sample slot (S) provided on the device.
- Step 4) Read the results between 5-10 minutes.
- Step 5) Discard the device after use

WHEN TO TEST

If the subject is still having monthly periods, then the test should be taken during the first week of her menstrual cycle (Days 2-7, with Day 1 being the first day of menstruation).

If the subject is no longer having regular periods, the first test should be taken at anytime during the month and the second test should be taken 1 week later. The result interpretation (refer the table below) will remain same for both the tests done in case of irregular periods.

PRECAUTIONS

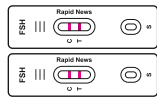

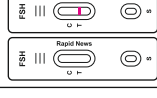
- The test should remain in the sealed pouch until use.
- Do not use if pouch is damaged.
- The test should be discarded according to local regulations.

SPECIMEN COLLECTION AND PREPARATION

The urine specimen must be collected in a clean and dry container. A first morning urine specimen is preferred since it generally contains the highest concentration of FSH; however, urine specimen collected at any time of the day may be used.

Back

RESULT INTERPRETATION

	<p style="font-weight: bold; color: #800080; font-size: 10px;">Positive</p> <p style="font-size: 8px;">Two pink lines at C & T. Line at T is equal or darker than C. It means that FSH levels are higher than normal. This means you are most likely experiencing perimenopause.</p>
	<p style="font-weight: bold; color: #800080; font-size: 10px;">Negative</p> <p style="font-size: 8px;">One pink line at C. Two pink lines at C & T. Line at T is lighter than C. It means that FSH levels are not elevated at this time. This means you are most likely not experiencing perimenopause in this cycle.</p>
	<p style="font-weight: bold; color: #800080; font-size: 10px;">Invalid</p> <p style="font-size: 8px;">One pink line at T. No pink lines at C & T. The test should be repeated with a new kit. This might occur when too little or too much urine was used. Follow the instructions carefully.</p>

PERIMENOPAUSE

- It is the stage that occurs in your early 40's and can extend for several years.
- There is a significant hormonal change, as the ovaries gradually cease functioning and estrogen production decreases.
- During this time, women commonly experience symptoms such as hot flashes, night sweats, irregular menstrual flow, and erratic menstrual cycles. These symptoms are a result of the fluctuating hormone levels and the reduced release of eggs.

MENOPAUSE

- It is the stage that is marked by the absence of menstrual cycle for 12 consecutive months, typically occurs in your late 40's to early 50's.
- During this stage, the ovaries cease to function, no longer releasing eggs.
- As estrogen and other hormone levels decrease, women may experience a range of physical and emotional changes, including hot flashes, mood swings, and changes in bone density.

Storage: Store at 2°C to 30°C. Protect from light & moisture. Do not freeze. Keep out of reach of children. For single use only. Not for internal or medicinal use.

This product is not intended to cure or prevent any disease. For in vitro diagnostic use only. FOR EXTERNAL USE ONLY

Presentation: Rapid News Menopause Test Kit contains 3N Test Kits are packed in a carton along with a package insert.

Manufactured in India by:
Mediforce Healthcare Pvt. Ltd., Unit-II
Plot No. 46, Industrial Area, Gondpur,
Tehsil Paonta Sahib, District Sirmour 173025,
Himachal Pradesh

Marketed by:
MANKIND PHARMA LTD.
208, Okhla Industrial Estate-3, New Delhi-110020
Contact Manager, Customer Care Cell:
Toll Free No.: 18001034400
Email: contact@mankindpharma.com
TM-Trade Mark Under Registration

MKCATPHNS1430500