


# Art Paper 80 GSM

Front 155.00 mm

Back



## Rapid News™

FOR HOME DETECTION

### Urinary Tract Infection Test

INSTANT URINE TEST KIT


**FAMILY HEALTH**

- + Easy to use
- + Instant results
- + Test from the convenience of your home

SYMPTOMS OF URINARY TRACT INFECTION

- + Sudden & intense urge to urinate.
- + Burning sensation while urinating.
- + Frequently passing small amount of urine.
- + Cloudy urine.
- + Urine that appears red, bright pink or reddish brown due to presence of blood.
- + Strong/unpleasant odour in urine.


URINARY SYSTEM

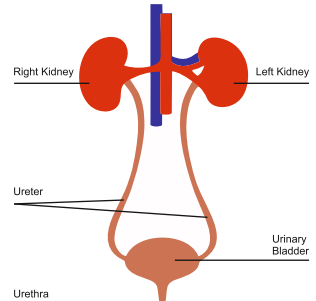


A urinary tract infection (UTI) is an infection in any part of the urinary system. The urinary system includes kidneys, ureters, bladder and urethra. Most infections in the urinary tract happen in the bladder & urethra.

If the UTI is limited to bladder, it can be painful. Serious health problems can occur if UTI spreads to kidneys. Hence it becomes critical to diagnose a UTI at an early stage.


PROCEDURE



Step 1  
Collect urine sample in a clean container





Step 2  
Dip the test strip in the urine sample



Step 3  
Check the results within 5 mins

- Step 1) Open the pack just before use.
- Step 2) Collect the urine sample in a clean and dry container.
- Step 3) Dip the test pads of the strip in the urine for 2 seconds and take out the strip.
- Step 4) Compare the colours of the test pad with the result chart.
- Step 5) Read the results within 5 minutes.
- Step 6) Discard the strip after use.

RESULT CHART

TOP TEST PAD :	NEGATIVE ←	→ POSITIVE
<b>Leukocytes</b> Compare the results with the chart after 2 minutes	-    +    ++    Any degree of purple    +++	
BOTTOM TEST PAD :	NEGATIVE ←	→ POSITIVE
<b>Nitrites</b> Compare the results with the chart after 1 minute	-    +    ++    Any degree of pink    +++	

Hold the test pad close to each colour block & match it with the closest colour block on the result chart (on the carton/pouch/leaflet). The colour on the test card indicates positive, negative result of leukocytes & nitrites. Check the table on back of the page to see what the result means.

**Leukocytes** are White blood cells, which are responsible for protecting your body from infection. Having leukocytes in the urine may be a symptom of an infection. A urinary tract infection (UTI) is often responsible for increasing leukocyte levels in the urine.

UTIs are caused by different types of bacteria. Many of these types of bacteria change a normal chemical in your urine, called nitrates, into another chemical, called **nitrites**. So, if you have nitrites in your urine, it usually means that you have a urinary tract infection (UTI).

### What do the results means?

Results		Recommendation
Leukocyte	Nitrite	
Negative	Negative	Results suggest no sign of UTI, if symptoms persists consult a physician.
Negative	Positive	Results suggest sign of UTI. Please increase intake of water and consult a physician.
Positive	Positive	Results suggest sign of UTI. Please increase intake of water and consult a physician.
Positive	Negative	Results suggest sign of UTI. Please increase intake of water and consult a physician.

Rapid News UTI Test Kit is only indicative. Results should be confirmed by a Registered Medical Practitioner. In case of positive result, consult a Registered Medical Practitioner.

DO'S

- 1) Wipe your intimate area from front to back after urinating.
- 2) Drink plenty of water so that you regularly urinate throughout the day.
- 3) Wash your intimate area with water before and after sex.
- 4) Urinate as soon as possible after sex.
- 5) Try to empty your bladder completely while urinating.

DON'T

- 1) Do not hold urine inside if you are feeling the urge to urinate.
- 2) Do not wear tight synthetic underwear.

**Storage:** Store at 2°C to 30°C. Protect from light & moisture. Do not freeze. Keep out of reach of children. This product is not intended to cure or prevent any disease. For single use only. For in vitro diagnostic use only. Not for internal or medicinal use. FOR EXTERNAL USE ONLY

Presentation: Rapid News UTI Test Kit contains 3N UTI Test Kits, packed in a carton along with a package insert.

Manufactured in India by :  
Mediforce Healthcare Pvt. Ltd., Unit-II  
Plot No. 46, Industrial Area, Gondpur,  
Tehsil Paonta Sahib,  
District Sirmour 173025,  
Himachal Pradesh

Marketed by:  
**MANKIND PHARMA LTD.**  
208, Okhla Industrial Estate-3,  
New Delhi-110020  
Contact Manager, Customer Care Cell:  
Toll Free No.: 18001034400  
Email: contact@mankindpharma.com  
TM-Trade Mark Under Registration

MKC5TPHTNS1406500